# Carestream

**DIRECTVIEW CR 500 Flexible Phosphor Screen GP**  
**DIRECTVIEW CR 500 Flexible Phosphor Screen HR**

**KODAK DIRECTVIEW CR 500 Flexible Phosphor Screen GP**  
**KODAK DIRECTVIEW CR 500 Flexible Phosphor Screen HR**

These products are for use in KODAK DIRECTVIEW CR 500 Systems.

Screen Warranty
The manufacturer warrants storage phosphor screens to function properly for one year from date of purchase, or from date of manufacture if proof of purchase is not available. This warranty applies only to new screens used in DIRECTVIEW CR 500 Cassettes, and does not apply to screens used in any automated or semi-automated screen-handling device not manufactured by the manufacturer. The manufacturer makes no other warranties, express, implied, or of merchantability, for these screens.
If these screens do not function properly within one year after purchase, or after date of manufacture if proof of purchase is not available, the manufacturer will replace the screens at no charge, unless the screens are damaged by misuse or other circumstances beyond the manufacturer's control.
<b>REPLACEMENT IS THE MANUFACTURER'S ONLY OBLIGATION. THE MANUFACTURER WILL NOT BE RESPONSIBLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES RESULTING FROM THE SALE OR USE OF THESE SCREENS, EVEN IF LOSS OR DAMAGE IS CAUSED BY THE NEGLIGENCE OR OTHER FAULT OF THE MANUFACTURER.</b>

#### Handling and Maintaining the Screen

Handle flexible phosphor screens with care. Hold screens by the edges and by the black side. Avoid contact with the screen's white (phosphor) side. Make sure hands are clean and dry. For best results, wear clean, soft, lint-free fabric gloves.

Under normal use conditions, flexible phosphor screens will eventually show wear. Screen wear can result in artifacts on radiographs. This wear may occur from abrasion of the protective overcoat or inadvertent physical damage to the surface.

**▲ CAUTION!** Screen damage can result from contact with certain materials used in facilities performing radiography. Contact with Isopropyl alcohol, peroxides, citrus-based cleaners, hand lotions and water-less hand sanitizers, as well as surfactants and lubricants may cause visible or hidden damage to the screen and could result in immediate or future image artifacts. **AVOID CONTACT BETWEEN THESE MATERIALS AND KODAK PHOSPHOR SCREENS.**

**▲ CAUTION!** Moisture can cause immediate or future screen damage and image artifacts. Minimize contact with moisture and **ALWAYS WIPE SCREENS DRY IMMEDIATELY.**

**▲ CAUTION!** Read and follow instructions in Material Safety Data Sheets (MSDS) for screen cleaner.

#### Cleaning the Screen

**IMPORTANT:** Inspect phosphor screens every week and clean every 200 exposures, every 30 days, or as necessary with a recommended cleaning solution used according to the manufacturer's instructions. Replace when wear is evident.

#### Cleaning Materials


- Clean, dry, non-abrasive, lint-free wipes/cloths.
- Screen cleaner from Carestream, CAT No. 845-4977.

**▲ CAUTION!** Cleaning solutions other than those recommended can contain chemicals that cause visible or hidden damage to the screen and could result in immediate or future image artifacts. Do not use Isopropyl alcohol (isopropanol, rubbing alcohol) to clean screens or cassettes.

#### Cleaning Method

- Place the screen with the black (non-phosphor) side down on a clean, dry, non-abrasive surface, such as the cassette's cleaned Tube Side.
- Clean and dry the screen's phosphor side.

#### Installing the Screen into the CR 500 Cassette

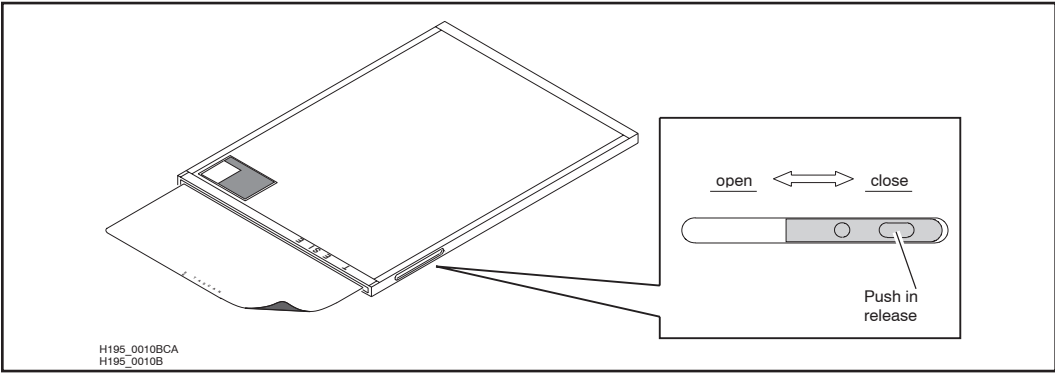**IMPORTANT!** Use the screen only with DIRECTVIEW CR 500 GP or HR Cassettes, available separately. Make sure to match the size and type (GP or HR) of screen to the size and type of cassette before inserting the screen.

- Place the cassette flat on a clean and dry horizontal surface with the Tube Side up.
- Place the tip of a tool, such as a small screwdriver or ballpoint pen, in the oval hole on the side of the cassette and push in to release the latch. Move the latch forward to open the cassette.
- Holding the cassette and screen horizontally, on a clean and dry flat surface, place the white (phosphor) side of the screen up so it faces the cassette's Tube Side panel. Carefully insert the screen all the way into the cassette.
- Place the tool into the round hole and pull the latch back. Move the latch forward to verify that the screen moves properly. Move the latch back as far as possible to close the cassette.

**▲ CAUTION!** If you fail to insert the screen correctly, calibration and functional failures may result:

- The scan calibration procedure may fail causing an error message.
- The screen may not feed properly from the cassette, causing an error message and preventing any exposed image on the screen from being processed.
- The screen may fall out of the cassette.

**▲ CAUTION!** Dilute bleach solution may cause eye irritation and dry skin. Wash hands with soap and water following use. Consult the manufacturer's Material Safety Data Sheets (MSDS) for additional information prior to use.

#### Disinfecting the Screen

- Wipe with a soft lint-free cloth thoroughly dampened (but not dripping) with 5% bleach solution (dilute one part of 5.25% sodium hypochlorite with nine parts of water). Contact the contaminated surface of the screen for up to 15 minutes for adequate disinfection.
- Wipe the bleach residue with a soft, lint-free cloth dampened with water to remove.
- Wipe dry with a soft, lint-free cloth. Allow screens to air dry thoroughly.

#### Disposing of the Screen

This screen, when discarded, is considered to be a hazardous waste (EPA waste code D005) under the Resource Conservation and Recovery Act (RCRA) due to the leachability of barium. Hazardous waste must be managed and transported in accordance with federal, state, and local regulations. Contact your local authorities for more information.

#### For Further Information

For further information concerning these products, inside the United States call Carestream Health, Inc. at 1-800-328-2910. Outside the United States, contact the Customer Support Center in your country.

CARESTREAM and DIRECTVIEW are trademarks of Carestream Health, Inc. The Kodak trademark and trade dress are used under license from Kodak.
--

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Documents concerning the conformance of this product to council Directive 93/42/EEC of 1993-06-14 concerning Medical Devices can be obtained from the Carestream Health European Representative at:

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**CE**  
**0086**

Carestream Health, Inc.  
150 Verona Street  
Rochester, NY 14608  
© Carestream Health, Inc., 2011  
P/N 9G6753 2011-05

**EC** **REP**

Carestream Health France  
1, rue Galilée  
93192 NOISY-LE-GRAND CEDEX  
FRANCE

**NEW P/N 9G6753** Back - 7 languages (English, German, Spanish, French, Italian, Brazilian Portuguese, Japanese)

**OLD P/N 6F8841A**

T.S. 18 in. x 16 in., 6 in. x 8 in.

Prints black on white stock

Dunn & Rice Design 05.06.11