Carestream

DIRECTVIEW CR 500 Cassette with GP Screen
DIRECTVIEW CR 500 Cassette with HR Screen

KODAK DIRECTVIEW CR 500 Cassette with GP Screen
KODAK DIRECTVIEW CR 500 Cassette with HR Screen

These products are for use with KODAK DIRECTVIEW CR 500 Systems.

Cassette Warranty

Carestream Health, Inc. warrants the DIRECTVIEW CR 500 Cassette to function properly for one year from date of purchase. Carestream Health makes no other warranties, express, implied, or of merchantability, for this cassette.

If the DIRECTVIEW CR 500 Cassette does not function properly within one year after purchase, Carestream Health will repair or replace the cassette, at its option and at no charge, unless the cassette is damaged by misuse or other circumstances beyond Carestream Health's control. Carestream Health will not repair or reimburse the customer for a cassette that is received with blood or bodily fluid contamination. Make sure that the cassette is decontaminated according to the Carestream Health Equipment Disinfection Procedure before shipment. The customer must include this signed declaration with the shipment of the cassette.

REPAIR OR REPLACEMENT IS CARESTREAM HEALTH'S ONLY OBLIGATION. CARESTREAM HEALTH WILL NOT BE RESPONSIBLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES RESULTING FROM THE SALE OR USE OF THE CASSETTE, EVEN IF LOSS OR DAMAGE IS CAUSED BY THE NEGLIGENCE OR OTHER FAULT OF CARESTREAM HEALTH. CARESTREAM HEALTH MAY USE OR SUPPLY RECONDITIONED, REMANUFACTURED, REURBISHED, OR USED PARTS AND COMPONENTS IN THE REPAIR OR REPLACEMENT OF CASSETTES UNDER WARRANTY.

IMPORTANT! Clean the phosphor screen before using the cassette for the first time!

- CAUTION! Failure to routinely clean the screens with a recommended solution may create the following failures as a result:
 - The screen may not transport from the cassette into the reader, causing an error message.
 - The screen may not fully return to the cassette at the end of the cycle, preventing the cassette from latching; no error message appears.

Cleaning the Screen

For all cleanings, clean screens with screen cleaner from Carestream, CAT No 845-4977. Follow the directions according to the manufacturer's instructions.


Read and follow instructions in the Material Safety Data Sheets (MSDS) for screen cleaner.

- Place the screen with the white (phosphor) side down on a clean, non-abrasive surface, such as the cassette's cleaned Tube Side.
- Following the specific product directions for the cleaning product you are using, clean the screen's entire non-phosphor side.
- After cleaning to the bottom, turn the screen over so the phosphor side is up.
- Following the specific cleaning product directions, clean the screen's entire phosphor side.

Other screen-cleaning agents may leave residues that will seriously affect the emission of these screens. The use of any cleaning agents other than those specifically suggested for cleaning flexible phosphor screens is not recommended.

CAUTION! Isopropyl alcohol (isopropanol, rubbing alcohol) and screen cleaners containing isopropyl alcohol are not recommended for cleaning storage phosphor screens. Isopropyl alcohol may contain organic peroxides, which can damage the screen.

Inserting the screen

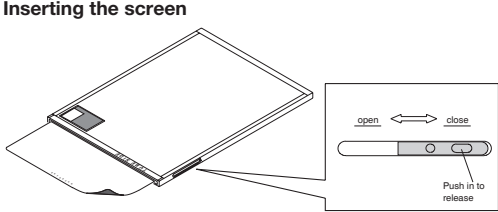CAUTION! Do not insert the screen with the cassette tipped at an angle.

- Place the cassette horizontally on a flat surface with the Tube Side up.
- Place the tip of a tool, such as a small screwdriver or ballpoint pen, in the oval hole on the side of the cassette.
- Push in to release the latch.
- Move the latch forward to open the cassette.
- Holding the cassette and screen horizontally, on a flat surface, place the white (phosphor) side of the screen up so that it faces the cassette's Tube Side panel.

NOTE: Use only DIRECTVIEW Flexible Phosphor Screens for CR 500 Systems, available separately. Match the screen resolution to the resolution indicated on the cassette label: (GP) General Purpose or (HR) High Resolution.

- Carefully insert the screen all the way into the cassette.
- Place the tool into the round hole and pull the latch back. Move the latch forward to verify that the screen moves properly. Move the latch back as far as possible to close the cassette. If you fail to insert the screen correctly, it could result in the calibration and functional failures listed below.

CAUTION: For proper operation and calibration of the 35 x 35 cm cassette, insert the screen into the cassette so that the black printing on the screen is positioned at the cassette door or cassette's back edge. It is possible to insert this screen into the cassette in any of four orientations.

If you fail to insert the screen correctly, the following failures could result:

- The scan calibration procedure may fail, causing an error message.
- The screen may not feed properly from the cassette, causing an error message and preventing any exposed image on the screen from being processed.

Cleaning the Cassette Shell (Exterior)
Remove the screen before cleaning the cassette shell. Clean the outer cassette surfaces using a soft cloth moistened with a mild soap-and-water solution or screen cleaner from Carestream. Do not use soaps or detergents containing brightening agents.

CAUTION! Isopropyl alcohol (isopropanol, not recommended for cleaning CR Cassettes. Isopropyl alcohol may contain organic peroxides, which can damage the phosphor screen.

CAUTION: Read and follow instructions in the Material Safety Data Sheet (MSDS) for screen cleaner.

CAUTION: When cleaning the cassette, avoid contaminating the inside of the cassette with the cleaning solution.

Image Area

Cassette Labeling

Cassettes are supplied with Bar Code and Screen Type labels on the cassette's non-Tube Side surface. You can apply the supplied Extremity label on this surface as well. Position it so that it does not interfere with the Bar Code label.

Loading the Cassette into the CR 500 Reader

- Hold the cassette so that the gray corners are towards you, Tube Side up, and bar code label is down.
- Insert the cassette into the CR 500 Reader with the engraved arrows pointed in the insertion direction.

Storing the Cassette

Store the cassette flat (Fig 1) or vertically on any long edge as shown in Fig 2. Do NOT store the cassette vertically with the screen exit slot facing down (Fig 3).

To avoid image artifacts caused by scatter radiation, do not store CR Cassettes and screens in the exam room during patient studies.

Cassette Disposal and Recycling
The CR Cassette contains lead. Disposal of lead may be regulated due to environmental considerations. For disposal or recycling information, contact your local authorities.

For further information
For further information concerning these products, inside the United States call Carestream Health, Inc. at 1-800-328-2910. Outside the United States, contact the Customer Support Center in your country.

CARESTREAM and DIRECTVIEW are trademarks of Carestream Health, Inc. The Kodak trademark and trade dress are used under license from Kodak.

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NEW P/N 9G7652A Front - 7 languages (English, German, Spanish, French, Italian, Brazilian Portuguese, Japanese)
OLD P/N 7H6239
T.S. 24 in. x 16 in., 6 in. x 8 in.
Prints black on white stock
Dunn & Rice Design 04.20.11

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© Carestream Health, Inc., 2011